

This is a lovingly crafted Temporary Mouth Mask (BMM). The Mouth and Nose Masks are not standardised and serve only as an aid. It was developed by over 277 volunteers sewing machines\* within the framework of the cup2gether® initiative are manufactured on a voluntary basis and not certified. The use of this product is at your own risk.



or



### \*Cleanse:

At least 1 x daily mask simply on the stove in a saucepan in boiling water 5 min. boil

Or: Wash at at least 60° degrees (WITHOUT detergent and not with the normal boil wash)

Dry in the dryer or on the line. Ironing at high temperatures.

### Benutzung:


The temporary mask should be max. 3 hours at the most.

Please change the mask if it is visibly dirty or wet.

After use, the Mask should be carefully removed and, if possible, not touch the Außenfl surface.

Wash your hands afterwards.

**Please continue to implement all protective measures consistently!  
Protect yourself and others!**

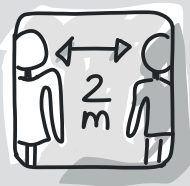

keep a distance



Sneeze and cough in your cubits



Hände regelmäßig mit Seife waschen



Do not touch the face (mouth, nose, eyes) with your hands and hand shoes

### General information:

Corona viruses are transmitted, for example, via droplets that are produced when speaking, coughing or sneezing and can reach the mucous membranes of contact persons.

The Mask can reduce the distribution of droplets into the environment and to contact persons. Especially when dealing with high-risk patients, it is useful to wear a BMM to minimize the distribution of droplets. It does not represent self-protection.

